
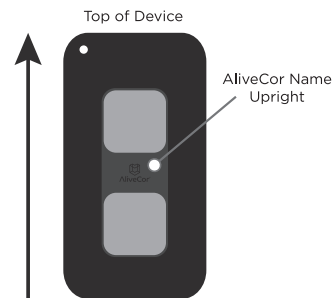


GETTING STARTED


HOW TO RECORD YOUR FIRST ECG

1 Meet Your AliveCor Mobile ECG

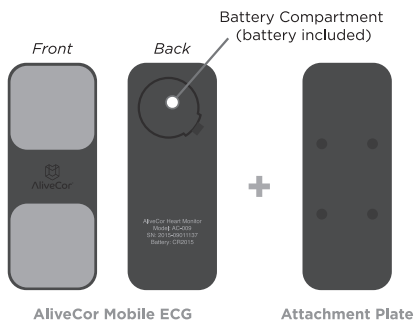


UNPACK

Take your AliveCor Mobile ECG out of the box. The Mobile ECG is already connected to an attachment plate that can adhere to the back of your smartphone or tablet or preferred device case.

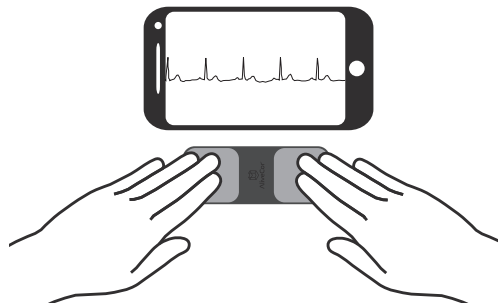

ATTACH (OPTIONAL)

Direction matters! Peel off the white strip from the back of the attachment plate, and stick the plate securely to the center back of your smartphone or tablet (or device case), checking to make sure that the word “AliveCor” in the center of the Mobile ECG is upright. In doing this, the electrode (silver plate) above the word “AliveCor” will be placed near the top of your device and the electrode below the word “AliveCor” will be placed near the bottom of your device.

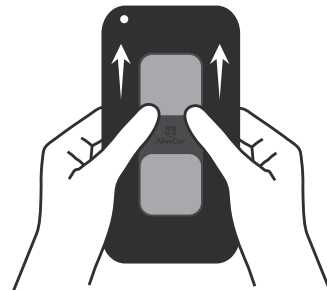

TIP: Some customers prefer not to attach the AliveCor Mobile ECG to their smartphone or tablet (or device case), which is just fine with us! Your Mobile ECG will work normally when it is placed within 12 inches (30 cm) of your smartphone or tablet.

TIP: If you buy a new smartphone or tablet or device case, your AliveCor Mobile ECG can go with you! To detach your Mobile ECG from the attachment plate, use your thumbs to slide the Mobile ECG toward the open end of the plate. Though the adhesive is designed for one-time use, additional attachment plates are available for a low price at store.alivecor.com in the U.S. or through one of our international distributors.




CREATE AN ACCOUNT

First, ensure your smartphone or tablet is connected to the Internet. Search for AliveECG in the Google Play™ store or App Store. Download and install the AliveECG app. If using an iPad, search for the AliveECG app for iPhone in the App Store.

TIP: If prompted during setup to enable the AliveECG app to access the microphone, please allow this access. Your AliveCor Mobile ECG works with the microphone on your smartphone or tablet.

Open the AliveECG app and follow the instructions to set up an AliveCor account. As a part of the account set-up, you will be asked to take your first recording.

Record Your First ECG

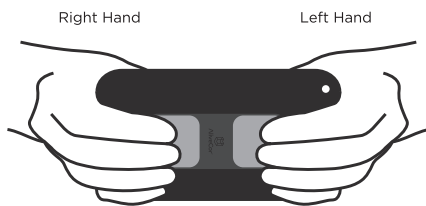

POSITION YOUR FINGERS

Place two or more fingers (it doesn't matter which fingers) from each hand on the electrodes for 30 seconds. While recording, try to relax your arms and hold still.

TIP: Each time you record, your left hand should touch the electrode above the word "AliveCor" and your right hand should touch the electrode below the word "AliveCor."



**Free Medical Interpretation
of Your First ECG Recording**
For the United States only

A U.S. board-certified cardiologist will automatically review your first recording for free and will provide a medical interpretation of your ECG within 24 hours. Due to FDA regulations, the heart rhythm for your first recording will not be visible on your smartphone or tablet screen and you will not be able to record any additional ECGs while the cardiologist is preparing your report. After you receive your report notification email, you will be able to record and view as many ECGs as you like.

If you have trouble with your AliveCor Mobile ECG, try the following support options:

 **IN THE UNITED STATES**

 www.alivecor.com/getstarted

 support@alivecor.com

 1-855-338-8800



OUTSIDE THE UNITED STATES

Please visit alivecor.com/getstarted for international customer support contact information.

AliveCor customer support is available Monday through Friday from 7 AM – 4 PM Pacific Time (PT).

00PL22.A